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Ergonomic Conization Electrodes

for Gynecology according to Scarfi

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What is a Conization and why is it Performed?

Conization is a surgical procedure during which a cone-shaped tissue specimen is excised from the portio uteri (vaginal part of uterus).

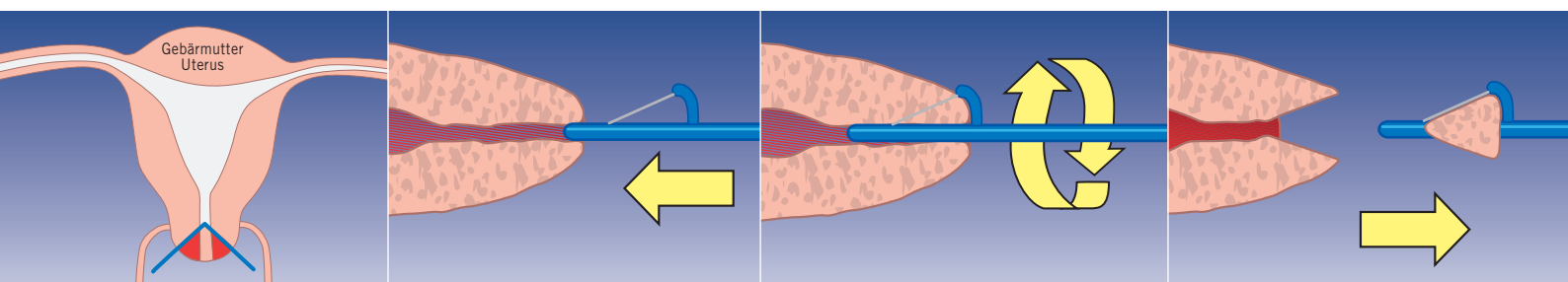
Tissue removal through conization is a safe method for diagnosing abnormal changes in the cervix. The result of the examination determines further treatment planning. In the very early stages of a malignant disease, conization can mean both diagnosis and sufficient therapy, especially for women who wish to have children.

How is a Conization Performed?

The procedure is performed through the vagina under general anesthesia. In order to rule out any further spread of pathological tissue into the interior of the uterus, a uterine cavity (cavum uteri) curettage is always performed as part of the conization. The removed cone is dyed in the laboratory, cut and examined under the microscope. Achieving tumor-free surgical margins is the key. Conization electrodes according to Scarfi ensure that optimal tissue is extracted for assessment by pathologists.


Surgical Technique

Compared with scalpel conization, high-frequency surgery with appropriate conization electrodes offers a smooth, low-hemorrhage cut. The electrode is introduced into the cervix upon activation of the high-frequency cutting current.

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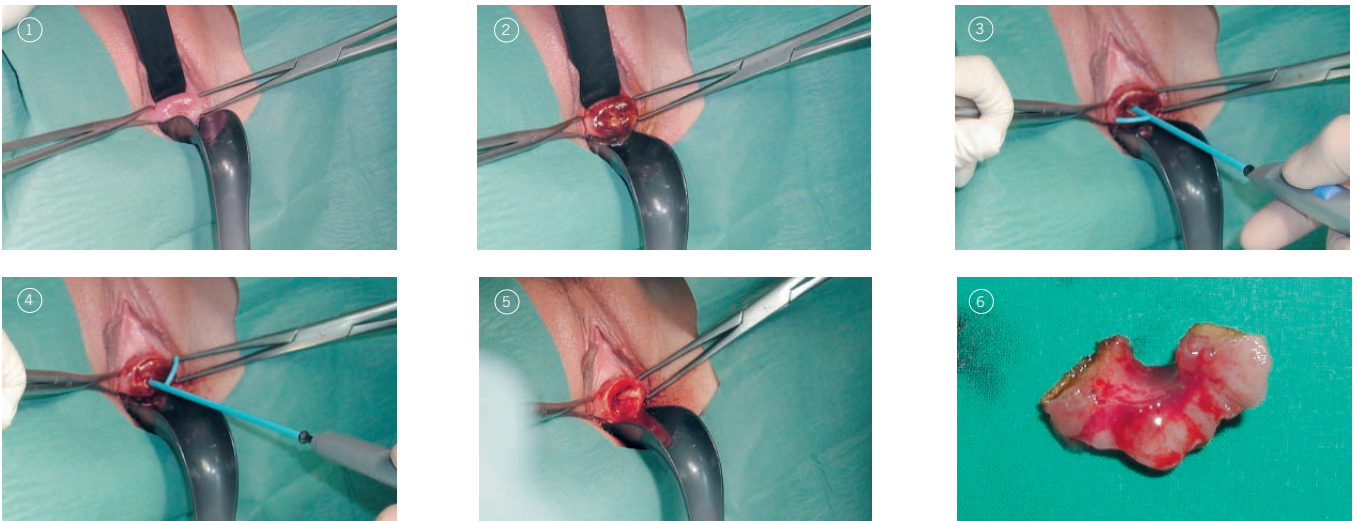The portio is optimally positioned with two bullet forceps. Using a standard electrode handle, it is possible to excise a defined cone with KLS Martin conization electrodes. All marTronic units of the ME series can be used for the cutting function.

The monopolar Pure Cut current is particularly suited to a smooth cut, which comes closest to the classic scalpel cut as the cut surfaces are coagulated only slightly. If heavy tissue bleeding is expected, a current type with medium hemostasis can be selected from the start (e.g. Blend Cut).

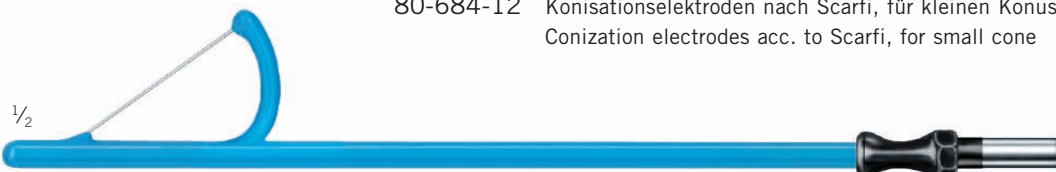
The conization electrode according to Scarfi is the ergonomic modification to a high-frequency gynecological instrument. The curved arm matches the shape of the tissue and thus perfectly adjusts to the portio.

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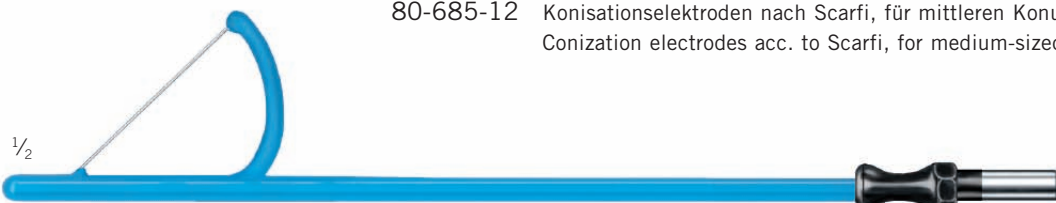
Osaka 541-0046
Japan
Tel. +81 6 62 28 90 75
nippon@klsmartin.com

Gebrüder Martin GmbH & Co. KG

Representative Office
121471 Moscow
Russia
Tel. +7 (499) 792-76-19
russia@klsmartin.com

KLS Martin L.P.

Jacksonville, Fl 32246
USA
Office phone +1 904 641 77 46
usa@klsmartin.com

Orthosurgical Implants Inc.

Miami, Fl 33186
USA
Office phone +1 877 969 45 45
sales@orthosurgical.com

Gebrüder Martin GmbH & Co. KG

A company of the KLS Martin Group
Ludwigstaler Str. 132 · D-78532 Tuttlingen
Postfach 60 · D-78501 Tuttlingen
Tel. +49 7461 706-0 · Fax +49 7461 706-193
info@klsmartin.com · www.klsmartin.com